



L a b t r o n C o .

Tel : +98 (21) 88899811-14 Fax : +98 (21) 88902493

P . O . Box : 15855-134 Tehran – IRAN

E-mail: info@labtronco.com www.labtronco.com

ELECTRONIC COUNTER

- **Easy to operate**
- **Automatic counts**
- **Memory for all readings**
- **Immediate recall for accumulated counts**



Micro- computer system for counting differential heamograms. Eight counter type with display and memory for all reading , fully automatic count. Immediate recall for accumulated counts and percentages at any stage.

INSTRUCTION FOR USE :

Switch on the mains and wait for automatic check; i.e. the display will show zero , the instruments ready for use.

To read the accumulated counts either use the Auto . , or 1...8 push – keys . Reset the instrument for the next count.

The memory is cleaned only, when reset is pressed or the instrument is switched off.

KEYBOARD EXPLANATION:

RESET: *This cleans the memory off.*

BEEP: *This is used for switching the buzzer on and off.*

DIS: *At any stage, pressing this key 1...8 will show on display the relevant accumulated number.*

AUTO: *This is used for automatic display of accumulated numbers for all channels ,which appears in sequence on display.*

1...8 : *These keys are used to enter the number of different W.B.C as stated under each key. Counts could be continued up to 999 and at any stage by pressing percentage key and the relevant key, the percentage of each W.B.C is obtained.*

SPECIFICATIONS :

Input Voltage: 220 v 50 Hz

Dimensions: 205 * 145 * 60 mm.

Weight: 650 g

Code No.: 3450